
# MAXITEST™ TIPO 5

## PT INDICADOR QUÍMICO - TIPO 5 INTEGRADOR VAPOR (STEAM)

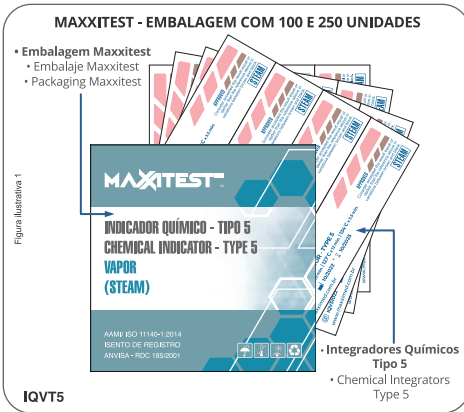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## EN CHEMICAL INDICATOR - TYPE 5 INTEGRATOR STEAM

### PRODUCT DESCRIPTION

The **MAXITEST CHEMICAL INTEGRATOR - Type 5** is a chemical indicator for internal use developed to carry out a complete and integrated evaluation of the critical parameters of the sterilization cycle (steam quality, exposure time and temperature). It promotes qualified reading of the conditions necessary for sterilization inside packages, boxes and containers.

In compliance with the AAMI/ISO 11140-1 standard, the product guarantees efficiency and safety in monitoring sterilization cycles. Its performance is similar to biological indicators, following the death curve of a theoretical population of *Geobacillus stearothermophilus* (ATCC 7953).

The **MAXITEST CHEMICAL INTEGRATOR - Type 5** is made of a medical grade paper strip, laminated front and back, and printed with a thermochromic chemical formula reacting to saturated steam. Provides result reading by color change that simplifies interpretation.

Dimensions: 19mm x 78mm (H x W). Presentation: boxes with 100 and 250 units.

### TECHNICAL INFORMATION

According to the determinations contained in the AAMI/ISO 11140-1 standard, the Type 5 chemical integrators must have their performance equivalent or superior to those presented by the biological indicators and also correlate with 03 sterilization cycles (temperature x exposure time) between 121 °C and 135 °C. The **MAXITEST CHEMICAL INTEGRATOR - Type 5** fully meets these performance requirements and includes the following sterilization cycles:

121°C	127°C	134°C
20 minutes	12 minutes	3,5 minutes

This product is an essential element in the composition of the Quality Management System in the reprocessing of materials and medical devices.

### INSTRUCTIONS FOR USE

1. Insert one unit of **MAXITEST CHEMICAL INTEGRATOR - Type 5** inside each package, box or container to be sterilized. Another **MAXITEST CHEMICAL INTEGRATOR - Type 5** must also be used in the assembly of a challenge test package that will work as a load release, according to ANVISA RDC 15: 2012.

2. Pack the load inside the auto-reading incubators' chamber, respecting the configuration approved in the validation (type of load, volume and weight).

3. Select and activate the sterilization cycle corresponding to the load to be sterilized.

4. At the end of the cycle, carefully and respecting the auto-reading incubators' cooling time recommended by the manufacturer, remove the **MAXITEST CHEMICAL INTEGRATOR - Type 5** from the inside of the challenge test package (if it has been used) and read the result.

**Satisfactory result and load released for storage in arsenal or for use in units:** when the chemical indicator ink dots change completely and uniformly from pink to brown tones to black color. Compare the change with the Interpretation Guide printed on the integrator and in this manual.

**IMPORTANT:** CONSIDER ACCEPTING THE CHEMICAL INTEGRATOR WITH RESULTS EQUAL TO OR DARKER THAN THE REFERENCE (BROWN OR BLACK COLOR SHADE).

**NOTE:** The color black means the process was successful. However, the exposure time was longer than required for efficient sterilization. These variations in thermochromic ink shades occur due to the heating curve of the autoclaves (heating ramp) until reaching the sterilization temperature. This heating time exposes the chemical integrators to sterilizing temperatures for a time much longer than the time-calibrated in a resistometer (ISO 11140). Thus, variations in shades of brown

and black are acceptable due to the non-uniformity of sterilization cycles in autoclaves and different load conditions.

5. If an **unsatisfactory result** is evidenced and the sterilization cycle fails, all materials contained in the load must be reprocessed.

6. The chemical indicator is a legal document. Therefore, its result must be recorded in an appropriate book/control and be attached to the patient's medical record, according to the routine established in the institution and health legislation in force.

**CAUTION:** Do not use the **MAXITEST CHEMICAL INTEGRATOR - Type 5** to monitor ethylene oxide sterilization cycles, dry heat, chemical vapors, radiation, or other sterilization processes that do not use saturated steam as a sterilizing agent.

### VALIDITY / ANVISA REGISTRATION

The **MAXITEST CHEMICAL INTEGRATOR - Type 5** is valid for 03 years from the date of manufacture. Lot information, expiration date and other identification data are printed on the packaging and on each unit of the product.

Product exempt from registration with ANVISA, according to RDC 185/2001.

### STORAGE

Store away from light at temperatures between 15°C and 30°C and relative humidity between 30% and 60%. Do not freeze. Do not store in contact with floors and walls or near heat sources, sterilizing agents and other chemical products.

### BIO PROGRAM

Green consciousness is the logic that drives MAXIMED's business model. For this reason, the company is proud to have its people, facilities, processes, products and services incorporated into a sustainable system.

### GREEN PARTNER PROGRAM

To ensure that the components used in its products are harmless to the environment, MAXIMED incorporates the "Green Partner" into its processes, a program that approves the company's suppliers and partners according to appropriate environmental parameters and encourages the conduct of business that generates value over time.

### LEAD FREE

MAXIMED uses lead-free technologies and other heavy materials that dissolve when in contact with acidic water, contaminating water, air and soil, seriously harming human health.

### FREE FROM VOLATILE ORGANIC COMPOUNDS

MAXIMED uses technologies free from Volatile Organic Compounds that, when exposed to closed places, can worsen human health.

These are some of the actions in which MAXIMED reinforces its commitment to continuous improvement and the preservation of the environment. Copyright 2019. Clicktest, Maxitest and Maxipack are registered trademarks of Maximed Latin America. Reproduction in whole or in part without written permission is prohibited.